



Clinical guidelines

Preparation techniques

With Procera AllZirkon you follow your normal procedures.

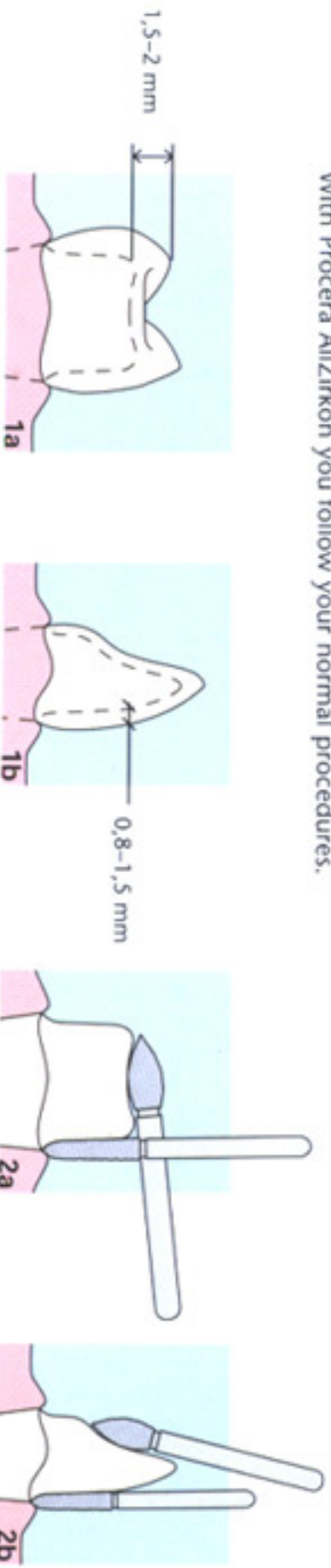

Recommended dimensions for preparation (figs. 1a and 1b). The preparation is characterized by thinned margins and a cutting with rounded inner angles and edges (figs. 2a and 2b).

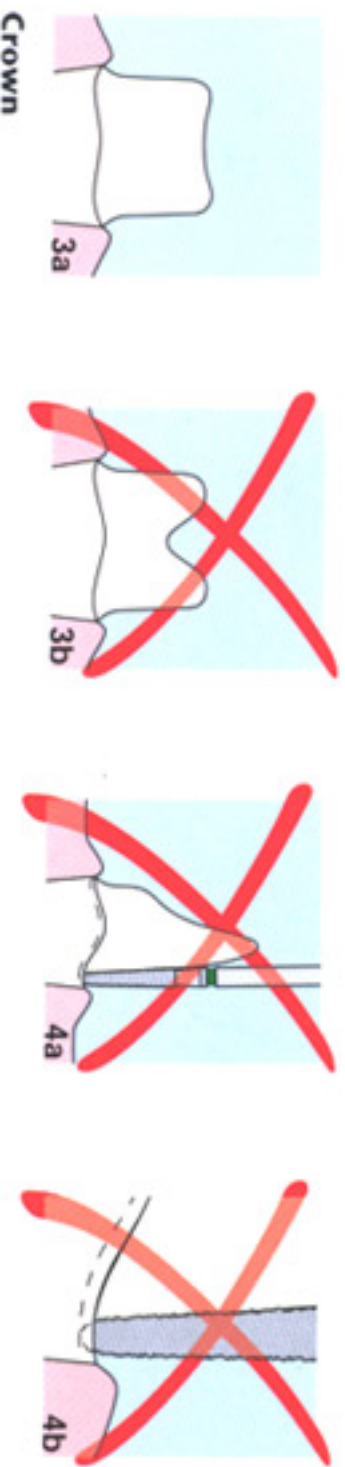

Crown

Keep the occlusal surface as flat as possible (fig. 3a) Do not create a deep cavity (fig. 3b) as this will make it difficult for the laboratory to scan the die. Make sure the apex of the diamond doesn't pass the edge of the margin and create a groove inside the margin (figs. 4a and 4b).